

Diversey™

Oxivir® Plus Spray



Broad Spectrum Cleaner & Disinfectant

Description

Diversey Oxivir Plus Spray is a ready to use cleaner disinfectant for use on hard, nonporous surfaces and some soft furnishings.

Key properties

- Patented Hydrogen Peroxide based acidic formulation
- Bactericidal, Virucidal, Fungicidal, Yeasticidal
- Short contact times
- Effective cleaner
- Fragrance free

Benefits

- Cleans and disinfects in one step, reducing working time
- Extensively tested against European norms and effective against multiple pathogens of concern, including HIV, HBV, HCV, Avian Influenza, Pseudomonas aeruginosa, Staphylococcus aureus, Salmonella enterica, MRSA, VRE and Norovirus
- Excellent cleaning performance, including descaling and lime limitation
- Good user safety profile, with no hazard warnings on RTU format; does not contain respiratory irritants
- Good environmental profile. Active breaks down into oxygen and water
- Suitable for use in high risk areas in Healthcare, including OR, ICU, NICU and patient rooms

Use instructions

Dosage:

Use undiluted

One Step Application:


Spray cleaning and disinfection:

1. Remove any gross soil
2. Spray onto surfaces
3. Wipe with clean cloth

Note: To disinfect for Norovirus, pre clean the surface prior to disinfection. Allow surface to remain wet for two minutes.

Important:

Do not mix with other products. Do not use on surfaces sensitive to acid.

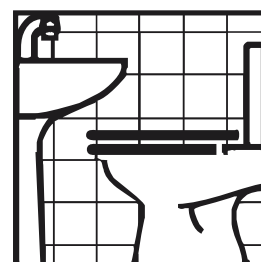



Oxivir[®] Plus Spray



Broad Spectrum Cleaner & Disinfectant

Technical Data

Appearance: Clear colourless liquid
Relative density [20° C]: 1.000
pH-value in use: 1.8

The above data is typical of normal production and should not be taken as a specification.

Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Material Safety Data Sheet. Store in original container. Avoid extremes of temperature. Reacts with oxydisable substances.

Product Compatibility

Under recommended conditions of use, Diversey Oxivir Plus Spray is suitable for use on most materials commonly encountered.

Microbiological data:

Efficacy proven by compliance with: EN1276, EN13697, EN1650, EN14776, EN13624 and EN13727.

Available pack sizes

Article code	Pack Size	Pack Type
7519553	Oxivir Plus Spray 6x0.75L	Spray bottle with foam trigger
7519554	Oxivir Plus Spray 6x0.75L	Spray bottle with foam trigger
7519559	Oxivir Plus Spray 6x0.75L	Spray bottle with foam trigger

Diversey South Africa (Pty) Ltd

Co. Reg. no: 2017/227398/07

Cnr Nagington & Rossouw Roads, Private Bag X037, 1422, Wadeville, Gauteng, South Africa.
Tel: (011) 871 9000 www.diversey.com

© 2017 Diversey. All Rights Reserved. Document Identification Number: 0327171310